
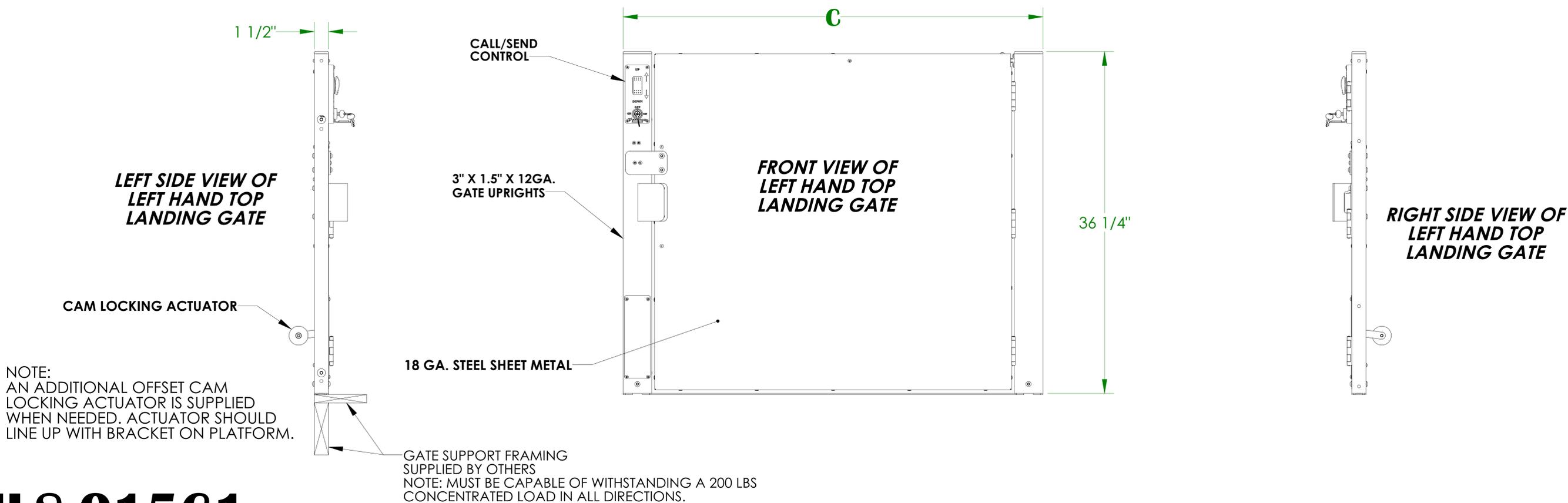


LANDING GATE DETAIL (RES)

VPL-3100/VPL-3200B SERIES VERTICAL PLATFORM LIFT (VIEWED AT TOP LANDING)

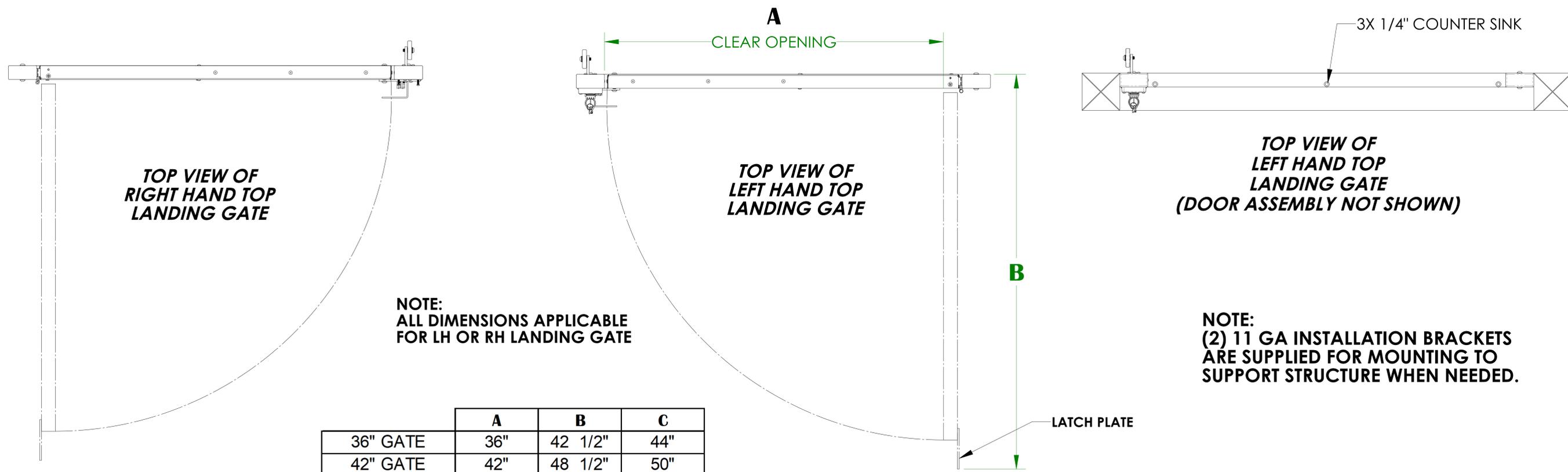

ILS-01561
SHEET 1 OF 3
REV. 0 (7683)(11/03/22)(TAK)

Bruno Independent Living Aids, Inc., 1780 Executive Drive, P.O. Box 84, Oconomowoc, WI 53066
 Phone (800) 882-8183 Fax (262) 953-5501

Any reproduction or other use of these materials without written permission of Bruno Independent Living Aids, Inc. is expressly prohibited. Bruno Independent Living Aids, Inc. reserves the right to modify or make changes to these specifications at any time without notice.



©Bruno Independent Living Aids, Inc.

LANDING GATE ALIGNMENT (STRAIGHT-THRU PLATFORM)(RES)

VPL-3100/VPL-3200B SERIES VERTICAL PLATFORM LIFT (VIEWED AT TOP LANDING)

RAMP
IN UP
POSITION

INSIDE OF PERIMETER TUBE TO
BE FLUSH WITH THE INSIDE WALL
OF THE GATE UPRIGHT TUBE ON
THE DOOR HANDLE SIDE.

***RIGHT HAND UNIT
WITH LEFT HAND
GATE SHOWN***

GATE SUPPORT FRAMING
SUPPLIED BY OTHERS
NOTE: MUST BE CAPABLE OF WITHSTANDING A 200 LBS
CONCENTRATED LOAD IN ALL DIRECTIONS.

GATE SUPPORT FRAMING
SUPPLIED BY OTHERS
NOTE: MUST BE CAPABLE OF WITHSTANDING A 200 LBS
CONCENTRATED LOAD IN ALL DIRECTIONS.

RIGHT HAND UNIT	E
LEFT HAND GATE	10 1/4"
RIGHT HAND GATE	8 1/2"

LEFT HAND UNIT	E
LEFT HAND GATE	8 1/2"
RIGHT HAND GATE	10 1/4"

***TOP VIEW OF PLATFORM
AND TOP LANDING GATE***

REFER TO SHEET 1 FOR GATE DIMENSIONS

DO NOT DRILL
THIS AREA

DO NOT DRILL
THIS AREA

***FRONT VIEW OF PLATFORM
AND TOP LANDING GATE***

3/8" MIN
3/4" MAX

***RIGHT SIDE VIEW OF
PLATFORM AND TOP
LANDING GATE***

***LEFT SIDE VIEW OF
PLATFORM AND TOP
LANDING GATE***

2"
FACE OF RAMP
TO GATE POST

BRACKETS TO BE ADJUSTED AS NEEDED
FOR PROPER ACTIVATION OF
CAM LOCKING ACTUATOR.

**ILS-01561
SHEET 2 OF 3**

REV. 0 (7683)(11/03/22)(TAK)

Bruno Independent Living Aids, Inc., 1780 Executive Drive, P.O. Box 84, Oconomowoc, WI 53066
Phone (800) 882-8183 Fax (262) 953-5501

Any reproduction or other use of these materials without written permission of Bruno Independent Living Aids, Inc. is expressly prohibited.
Bruno Independent Living Aids, Inc. reserves the right to modify or make changes to these specifications at any time without notice.



©Bruno Independent Living Aids, Inc.

LANDING GATE ALIGNMENT (90° / ADJACENT EXIT PLATFORM)(RES)

VPL-3100/VPL-3200B SERIES VERTICAL PLATFORM LIFT (VIEWED AT TOP LANDING)

**RIGHT HAND UNIT
WITH LEFT HAND
GATE SHOWN**

**NOTE:
LEFT HAND GATE MUST BE USED
WITH RIGHT HAND UNIT.
RIGHT HAND GATE MUST BE USED
WITH LEFT HAND UNIT.**

INSIDE OF PERIMETER TUBE TO
BE FLUSH WITH THE INSIDE WALL
OF THE GATE UPRIGHT TUBE ON
THE DOOR HANDLE SIDE.

RAMP
IN UP
POSITION

GATE SUPPORT FRAMING
SUPPLIED BY OTHERS
NOTE: MUST BE CAPABLE OF WITHSTANDING A 200 LBS
CONCENTRATED LOAD IN ALL DIRECTIONS.


GATE SUPPORT FRAMING
SUPPLIED BY OTHERS
NOTE: MUST BE CAPABLE OF WITHSTANDING A 200 LBS
CONCENTRATED LOAD IN ALL DIRECTIONS.

1 3/4"

OUTSIDE OF PLATFORM
TO GATE SUPPORT FRAMING
SUPPLIED BY OTHERS

**TOP VIEW OF PLATFORM
AND TOP LANDING GATE**


REFER TO SHEET 1 FOR GATE DIMENSIONS

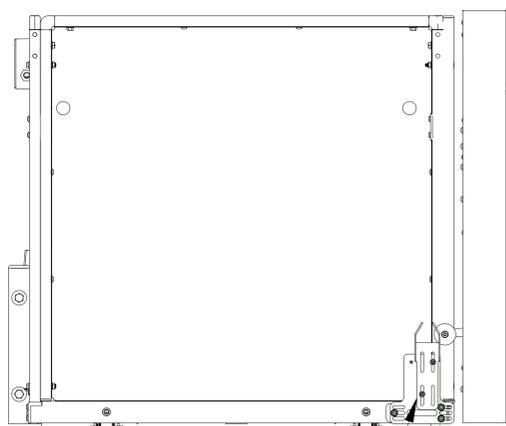


**LEFT SIDE VIEW OF
PLATFORM AND TOP
LANDING GATE**

DO NOT DRILL
THIS AREA

DO NOT DRILL
THIS AREA

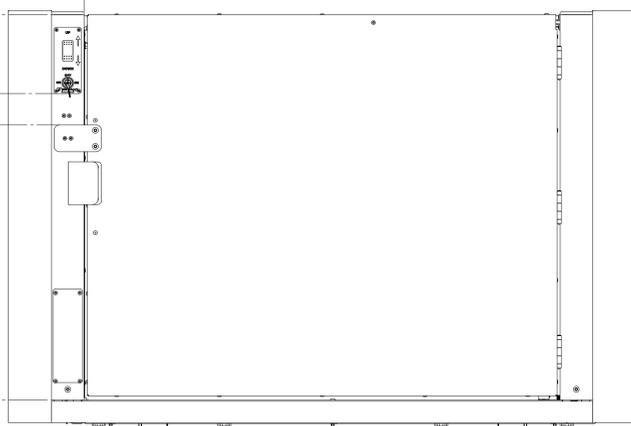

**FRONT VIEW OF PLATFORM
AND TOP LANDING GATE**

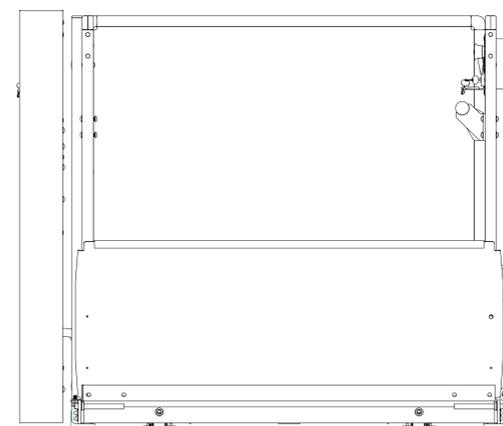

**RIGHT SIDE VIEW OF
PLATFORM AND TOP
LANDING GATE**

2"
FACE OF RAMP
TO GATE POST

3/8" MIN
3/4" MAX

BRACKETS TO BE ADJUSTED AS NEEDED
FOR PROPER ACTIVATION OF
CAM LOCKING ACTUATOR.

Bruno Independent Living Aids, Inc., 1780 Executive Drive, P.O. Box 84, Oconomowoc, WI 53066
Phone (800) 882-8183 Fax (262) 953-5501

Any reproduction or other use of these materials without written permission of Bruno Independent Living Aids, Inc. is expressly prohibited.
Bruno Independent Living Aids, Inc. reserves the right to modify or make changes to these specifications at any time with out notice.



©Bruno Independent Living Aids, Inc.

**ILS-01561
SHEET 3 OF 3
REV. 0 (7683)(11/03/22)(TAK)**